# DELABIE

# BIOFIL

**NT 30050**

Indice H

02-2025

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## 1 DESCRIPTION

The spout filters covered by this installation guide are all-germ filters that can be used instead of stainless steel or disposable spouts on DELABIE removable taps (BIOCLIP). They are commonly referred to as "**BIOFIL spout filters**". Their purpose is to filter bacteria present in drinking water systems.

They are installed at the end of drinking water systems on taps designed for this purpose. They serve as **physical barriers in the event of a bacterial alert** (*Legionella pneumophila*, *Pseudomonas aeruginosa*, and other Gram-negative bacteria, non-tuberculous Mycobacteria, fungi, protozoa and other particles from the water distribution system – absolute filtration grade of 0.2 µm) for a duration of use of **2 months** after installation.

These are **single-use** filters that can be used for washing hands and medical equipment. They are intended for controlled environments only such as hospitals and other medical institutions.

## 2 REFERENCES

- Non-Sterile BIOFIL spout filter: 30040.
- Sterile BIOFIL spout filter: 20040.

**To guarantee optimal performance of the BIOFIL filters, please comply with the installation protocol (see 3) and the precautions for use (see 7).**

A complete list of filters is available:

**For the UK market only:** on our website [www.delabie.co.uk](http://www.delabie.co.uk) or from our customer service team: Tel. 01491 821821 - Email: [technical@delabie.co.uk](mailto:technical@delabie.co.uk). **For all other markets:** on our website [www.delabie.com](http://www.delabie.com) or from our customer service team: Tel. +33 (0)3 22 60 22 74 - Email: [sav@delabie.fr](mailto:sav@delabie.fr)

## 3 INSTALLATION

### Warnings:

Wash and disinfect your hands before installing the filter to limit the risk of contamination or retro-contamination. Check that the individual pack is not punctured or torn and that the filter is not damaged.

Discard the filter if this is the case.

Check that the expiry date has not passed. Discard the filter if this is the case. Dispose of the filter as ordinary household waste.

### Preliminary preparation of the taps:

- The spout filters are used instead of stainless steel or disposable spouts on DELABIE removable taps (BIOCLIP). They filter bacteria commonly present in drinking water systems.

### Installation:

1. Remove the stainless steel or disposable spout from the DELABIE BIOCLIP tap (no tools necessary) and disinfect the outlet according to current procedures in place.
2. **Wash and disinfect your hands.**
3. Partially open the unit packaging.
4. Note the commissioning and the theoretical decommissioning date (installation date + theoretical duration of use) on the filter and on the traceability label.
5. While keeping the filter in the bag to avoid contamination by the water outlet, place the opening of the spout at a 45° angle to the tap and fit the spout while ensuring it can rotate freely.
6. Once it has reached a stop, rotate the spout a few degrees until it stops again, then pull the spout towards you to lock it.
7. Remove the packaging.
8. Turn the water on for a few seconds to check that the filter is completely watertight.
9. If there is a leak, discard the filter in the household waste and start again. Do not reuse any filter that has been connected.

### Disposal after use:

1. Remove the spout by unclipping it from the tap.
2. Dispose of the filter as ordinary household waste. Never reuse a filter that has been connected. Because the filter is attached to the spout, the manufacturer cannot guarantee that the fibres are still in good condition.

Installation and use in a dirty or uncontrolled environment can be a source of retro-contamination. Discard and replace any filter that appears visually soiled or contaminated.

## 4 CLEANING

The outer surfaces of the filter, except for the water outlet, can be cleaned with a single-use disinfectant wipe following the current hygiene procedures in place (wearing PPE, specific protocols, etc.)

## 5 TECHNICAL PERFORMANCE

### Conditions of use:

The performance of BIOFIL filter spouts is guaranteed when the pressure and temperature of the water in the system are between 1 and 5 bar and 5°C and 42°C respectively. The water flow rates from the spout (filter only, without flow restriction in the tap) measured at the outlet are as follows:

	BIOFIL filter spout				
Pressure (bar)	1	2	3	4	5
Flow rate (l/min)	4	5	6	8.5	9.5

## Exposure to chemical and thermal treatments:

The filter can tolerate the following treatments:

- Heat treatment of 70°C for 30 minutes (maximum cumulative).
- Chemical treatment: Chlorine shock at 100 PPM for 30 minutes at a flow rate of 0.6 l/min.  
The filter should be replaced following either of these treatments.

## Clogging tolerance:

If the water in the drinking water system has an unusually high level of particles or microorganisms in suspension, it is possible to observe a decrease in the outlet flow rate before the theoretical decommissioning date (installation date + theoretical duration of use).

When the flow rate becomes too low, replace the filter.

**Warning:** the filter may get clogged before the theoretical decommissioning date.

**Advice:** if the filter clogs prematurely, upstream pre-filtration at different levels of the system is recommended to stop suspended solids and/or colloids.

## 6 DISPOSAL

Used filters do not pose a risk of infection. Like all bi-material packages, they are classified as household waste. The instructions and the cardboard box are recyclable (paper, cardboard).

## 7 PRECAUTIONS FOR USE

- Wash and disinfect your hands before touching or cleaning a filter.
- Purge the tap with the removable spout in place before installing the filter to avoid early clogging.
- Do not use the filter if the packaging and/or the filter are damaged.
- Avoid collisions or bumps with the filter. Replace if damaged.
- Discard and replace any filter that appears visually soiled or contaminated.
- Do not disconnect the filter while in use.
- The water outlet must not come into contact with skin or rinsed equipment. It is advisable to replace the filter if contact has occurred.
- If there is a decrease in water flow, it is advisable to replace the filter.
- Once disconnected, the filter should be discarded (single use).
- The microfiltration cartridge must not be adapted, modified or opened.
- The filter must not be used for the production of water for pharmaceutical use.
- The filter is not intended to produce sterile water.
- The filter does not change the potability of the water, and it does not modify its concentration of salts, metals, scale, chemicals, pesticides or dyes.

## 8 PRODUCT COMPLIANCE

BIOFIL filter spouts have received an ACS drinking water compliance certificate in France.

## 9 MANUFACTURER DETAILS

BIOFIL filters are manufactured by:

**AQUATOOLS** | 11, rue Lobineau | 75006 PARIS | France

Website: [www.aqua-tools.com](http://www.aqua-tools.com)

And are distributed by DELABIE:

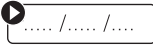
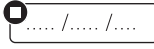
### 1. For the UK market only:

visit our website [www.delabie.co.uk](http://www.delabie.co.uk) or contact us: Tel. 01491 821821 - Email: [technical@delabie.co.uk](mailto:technical@delabie.co.uk)

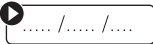
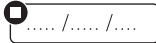
**For all other markets:** visit our website [www.delabie.com](http://www.delabie.com) or contact us: Tel. +33 (0)3 22 60 22 74 - Email: [sav@delabie.fr](mailto:sav@delabie.fr)

### 2. A paper version of the installation guide is available on request.

## Product label - Symbol key - Non-sterile - 30040

		
Catalogue reference	Batch code	Expiry date
		
Do not use if the packaging is damaged	Manufacturer's address	Non-sterile
		
Do not reuse	Consult the precautions for use	Commissioning date
		
Decommissioning date	Use-by date after commissioning - 2M: 2 months	

**Product label – Symbol key – Sterile – 20040**

		
<p>Catalogue reference</p>	<p>Batch code</p>	<p>Expiry date</p>
		
<p>Device sterilised by irradiation</p>	<p>Do not resterilise</p>	<p>Do not use if the packaging is damaged</p>
		
<p>Manufacturer's address</p>	<p>Manufacture date</p>	<p>Consult the precautions for use</p>
		
<p>Sterile barrier system only</p>	<p>Do not reuse</p>	<p>Commissioning date</p>
		
<p>Decommissioning date</p>	<p>Use-by date after commissioning - 2M: 2 months</p>	

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